



heartisense™ Quick Sales Guide

HeartiSense makes existing CPR manikins much smarter!

Experience a high quality CPR training by attaching HeartiSense Kit to your CPR manikin

HeartSense Product Components

HeartiSense Kit



HeartiSense Kit Sensing device that measures compression(position, depth, rate) and respiration(breath volume) during CPR training

Kit Components

Control Module


Sends data from each sensor to tablet PC

Compression Pad

Measures the position, depth, rate of chest compression

Breath Module

Measures breath volume

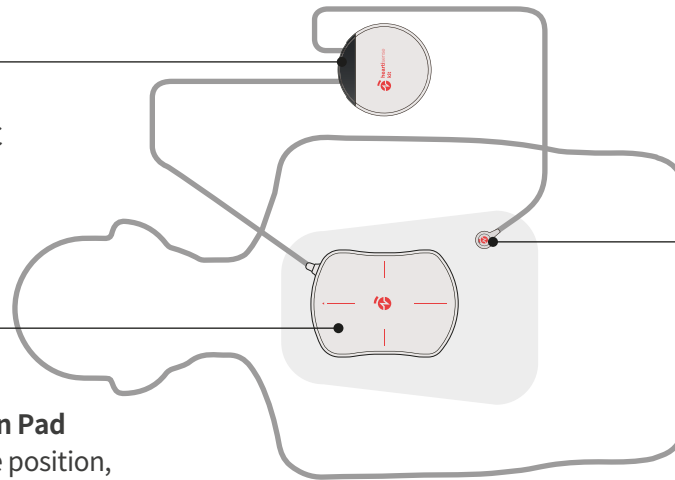

Kit Types


Basic (For Student app) / Premium (For Student app + Instructor app)

HeartiSense App (iOS / Android)



HeartiSense Student

Application for trainees that can receive real-time feedbacks for themselves

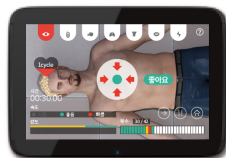

HeartiSense Instructor

Application that can train, evaluate and check results up to six trainees simultaneously



HeartiSense LMS







HeartiSense LMS

Online data managing system that can manage training and evaluation data of the students

*Only available during the use of HeartiSense Instructor



HeartiSense Product Line

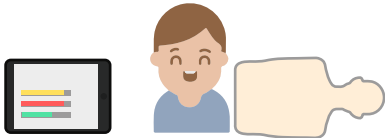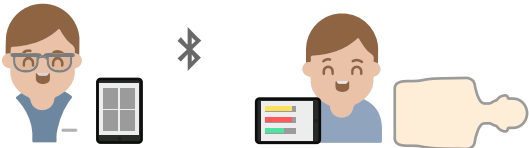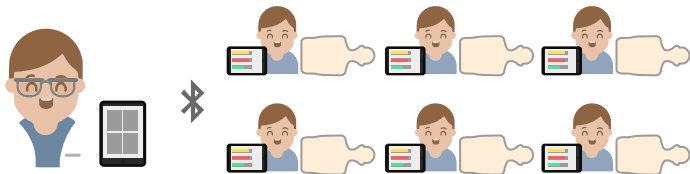HeartiSense Product Line	
Function	Components
<p>HeartiSense Basic</p> <p>A trainee can do CPR training by receiving real-time feedbacks without an instructor</p> 	<p>Includes</p> <p>HeartiSense Basic Kit (1 ea),</p> <p>HeartiSense Student app</p>
<p>HeartiSense Premium</p> <p>An instructor can have an effective personal training and evaluation by giving high quality CPR feedbacks</p> 	<p>Includes</p> <p>HeartiSense Premium Kit (1 ea),</p> <p>HeartiSense Student app ,</p> <p>HeartiSense Instructor app</p>
<p>HeartiSense Premium 6 Pack</p> <p>An instructor can have an efficient group training and evaluation simultaneously (Max of 6) by giving high quality CPR feedbacks</p> 	<p>Includes</p> <p>HeartiSense Premium Kit (6 ea) ,</p> <p>HeartiSense Student app ,</p> <p>HeartiSense Instructor app</p>
<p>HeartiSense LMS</p> <p>Can only be used in HeartiSense Premium, it can be purchased on annual license basis</p> 	<p>HeartiSense LMS license</p> <p>(1year, 600data)</p>


* Recommendations

HeartiSense is comprised of Kit and App, and does not include a tablet or a training manikin. If a distributor wishes to create additional packages including a tablet or a training manikin, we recommend the following products.


[Tablet] Bluetooth 4.0 or more recent versions
 Android : 4.3 or more recent (Asus Google new Nexus 7, Galaxy Tab)
 iOS : iOS 9.0 or more recent (iPad, iPad mini)

[Manikin] Skin type manikin (Laerdal Little Anne, Prestan, Brayden etc.)


Simple Installation Directions of HeartiSense

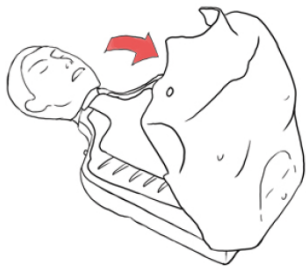


Remove the skin of the manikin.

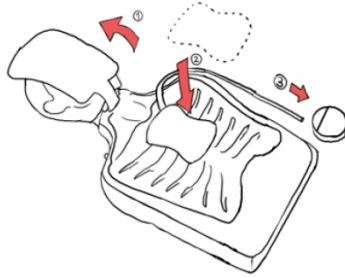

Attach the compression pad on the center of the chest plate which is located under the lung bag of the manikin and connect it to the control module.

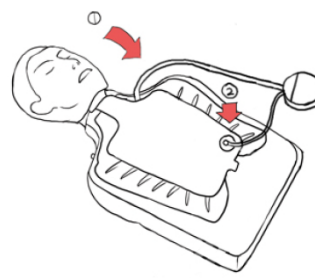

Attach the breath module to the lung bag and connect it to the control module.

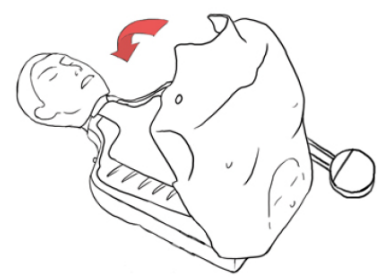

Cover the manikin with the skin and complete HeartiSense kit installation.

How to Use HeartiSense Application (iOS / Android)





Access into the market (iOS: AppStore / Android: PlayStore) and search "HeartiSense". Download the application.



Open the application and connect the application with the HeartiSense kit by Bluetooth. Start the training and assessment using the app

How to Use HeartiSense LMS

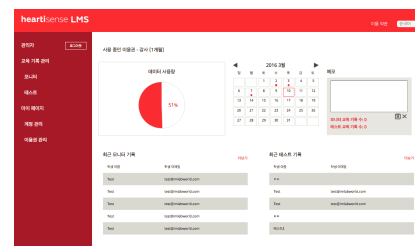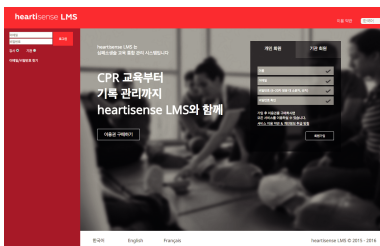

You can send HeartiSense Instructor results data to the LMS server when connected to Internet. You can open and manage students information within the LMS homepage.

Product information (www.heartisense.com)

Manual and instruction guide
(www.heartisense.com/support/userguide.php)

Contact info

I.M.LAB Inc.

Address 302, 87 Daehak-ro, Yuseong-gu, Daejeon, South Korea(34167)

Tel (+82) 70-5056-3900 Fax (+82) 70-8230-3901

Homepage www.imlabworld.com

E-mail contact@imlabworld.com